

The indicator monitors temperature exposure, not product quality

Vaccine Temperature Management

A ColdMark™ Freeze Indicator has been packaged with this shipment by McKesson to assure that the enclosed vaccines have been shipped under manufacturer-recommended conditions.

Receiver Instructions

Immediately upon shipment receipt:

- Remove the ColdMark Freeze Indicator from the shipment container
- Store vaccines as instructed below
- Follow instructions on reverse side to utilize the ColdMark Freeze Indicator properly

Vaccine Storage Requirements

- Take the product out of the shipment container immediately upon shipment receipt
- If damage is evident, contact your Project Point of Contact (PPOC) within 2 hours of shipment receipt
- Remove the product from the zippered plastic bag
- Refrigerate immediately
- Store at 35° to 46°F (2° to 8°C)
- Do not freeze or expose to freezing temperatures

How to Read

Immediately upon shipment receipt, remove the ColdMark Freeze Indicator and check the ColdMark bulb.

- If the bulb is clear and colorless, begin using vaccine as needed.



- If the bulb appears violet in color, store the product as instructed and contact your PPOC for further instructions prior to using.



ColdMark Freeze Indicator decision table:

Bulb Color	Vaccine Type	
	MMR, ActHIB, LAIV	All other vaccine
Clear and colorless	Begin using vaccine	Begin using vaccine
Violet	Begin using vaccine	Contact PPOC